

## Filling Instructions\*

Read all information, specifications and instructions carefully before filling and dispensing in Topi-CLICKs.

### Step 1



Fill Topi-CLICK with desired amount using Topi-CLICK Filling Station, or EMP jar & nozzle, or spatula.

### Step 1a



If necessary, stop every third of the fill to tap the Topi-CLICK on the palm of your hand or counter to help settle the topical.

### Step 2



Snap applicator pad (cap with hole) on Topi-CLICK tube by aligning inside key tabs with slots on top-edge of tube and snap tight.

### Step 3



Prime the Topi-CLICK by turning the bottom until topical is dispensed and wipe off topical.

### Step 3a: LEAVE IN HOME / START POSITION



Priming: When rotating base, watch the Visual Indicators

Indicator Node on body is fixed at the CLICK point



Indicator Column moves with Base. This shows Node in-line with column.

The HOME / START POSITION

**VERY IMPORTANT TO LEAVE IN HOME POSITION WHICH IS THE START POSITION OF NEXT DOSE**

### Step 4



Snap on cover cap. (if cover cap doesn't fit tightly, it indicates applicator pad in Step 2 is not snapped all the way down).

### Step 5



Apply center of label to side of the Topi-CLICK container opposite of the "REFILL" and graduation marks.

For more information on dispensing amounts per click see the [Specifications](#) document or see [www.Topi-CLICK.com](http://www.Topi-CLICK.com). See the [User Instructions](#) sheet on how to apply with Topi-CLICKs.

Notes: <sup>1</sup>Each CLICK (¼ turn of the Topi-CLICK base) moves the plunger up approximately 0.25ml inside tube.

<sup>2</sup>Allow for extra topical needed for priming and 2ml residual.

<sup>3</sup>The graduation marks are for reference only & can be used with the bottom of the plunger.

<sup>4</sup>Some bases are not practical to use with the Topi-CLICK.

\*As required for compounded prescription pharmaceuticals; verify formulas and weight per volume.

For more information and to download this page see [www.Topi-CLICK.com](http://www.Topi-CLICK.com).

